



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Honey Gauze “Manuka” Dressing

4”x 4” (10 x 10 cm)
10/box
50/case
GEN-16700

- Gauze dressing impregnated with 100% Leptospermum (Manuka) Medical Grade Honey
- Easiest way to apply Medical Grade Honey
- Helps promote moist healing in challenging wounds and burns
- Assists in autolytic debridement



Gentell Honey Gauze Dressing is a primary wound dressing for partial- and full-thickness wounds, leg ulcers, pressure ulcers, first and second degree burns, diabetic foot ulcers, minor abrasions, and lacerations.

Directions

1. Flush the wound with Gentell Wound Cleanser, and gently dry the skin surrounding the wound site.
2. Peel open pouch and remove the Honey Gauze Dressing.
3. Remove the plastic from both sides of the Honey Gauze Dressing, and apply the dressing to the wound bed.
4. Apply cover dressing such as Gentell Foam Dressing, Gentell Bordered Gauze or Gentell Comfortell.
5. Repeat daily or as ordered by a physician.